



MEDICAL-X



# IV Sleeve

Task Trainer

Intravenous injection trainer sleeve

## Wearable IV Sleeve Trainer

This wearable IV sleeve trainer provides safe, real-life clinical training and can be used on standardized patients, or manikins.


The internal tool offers a realistic vein simulation that allows users to perform IV injection.

This task trainer simulator is made of a layer of silicone skin that creates a sensible experience of touch for the user.





## How to use it

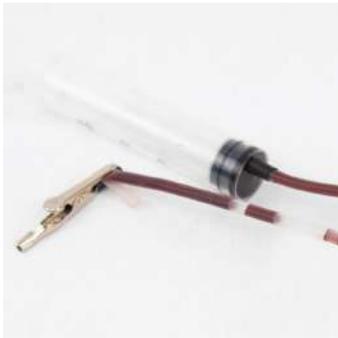

**Step1:** Cut the tube to the length needed. Use the syringe to fill the tube with simulated blood. Use the clamp to close the other end to prevent the blood to spill.



**Step2:** After the tube is filled with simulated blood, then close the other extremity with the second clamp.



**Step3:** Place the sleeve on a rigid support.



**Step3:** Place the tube on the sleeve keeping both exteremities outside.



**Step4:** Cover the tube with the silicone support.




**Step5:** Cover the silicone support with the hard plastic for protection of the arm.



**Step6** Place the arm on top of the support.



**Step7:** Close the sleeve adjusting to the size of your arm with the velcro.

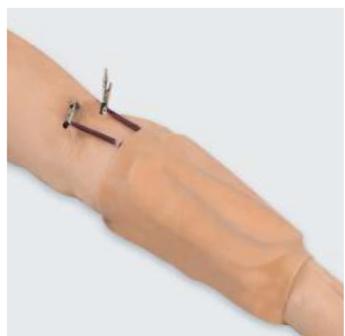

**Step8:** Practice IV injections on the veins. A new tube can be cut for every usage.


## This package comes with:



**IV Sleeve:** Reinforced silicone skin.



**Skin and Protection:** A silicone and plastic support to simulate the veins in the best way.

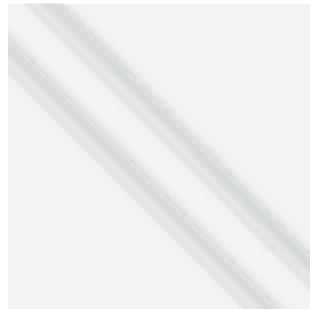

**Tube:** Disposable tube (180 cm ca.) Cut for use.



**Syringe and two clamps:** Syringe used to inject simulated blood and two clamps to close off the tube.



## Features List

### Dimensions

- 12 cm (4,7 Inch) in diameter.
- 24 cm (9,4 Inch) in length.

### Specifications

- The model can be used repeatedly.
- No needle included.
- Disposable tubes

### Materials

Silicone representing soft tissue and plastic for protection against the needle for the standardized patient or manikin.



## CONTACT US

Contact us for more information about our product and the ordering process.

 [www.medical-x.com](http://www.medical-x.com)

 [info@medical-x.com](mailto:info@medical-x.com)

 +31 10 341 5927

 Schiedamsedijk 77, 3011 EM  
Rotterdam, The Netherlands